

# CARE INSTRUCTIONS

Lia breast forms are manufactured with the highest quality materials and superior craftsmanship. To ensure that it maintains its soft supple feel and appearance, please give your form the same special care you give your body.

*To protect your form:*

- ◆ Store in its specially designed cradle.
- ◆ Avoid sharp objects and direct sunlight. Before handling your form, remove your rings and bracelets. Beware of small punctures caused by scissors, pins, jewelry and pet's claws or teeth.

Clean daily by gently washing your form using warm water and mild soap. Rinse thoroughly and pat dry with a clean cloth. Never wring, rub, use abrasive cleaners, acetone, alcohol or any other solvent on your breast form. Powders will tend to dry form, while perfumes may damage the protective outer layer.

**NOTE: DO NOT HOLD/PICK UP FORM WITH YOUR FINGERS.** Hold form with the palm/cup of your hands. This will prolong the wear of you Lia breast form.